



Turner transfer aid

The Turner is a practical aid which activates the patient when transferring from one seated position to another. It offers security for the person being transferred and will not strain the carer's back.

Easier transfers

Many often feel unsafe when being assisted from their beds to their wheelchairs, for example, and carers are subject to strain when lifting and turning them. With the Turner, the patient controls how he or she stands up. This reduces the risk of falling, and it also allows the carer to work in an ergonomically correct manner. Only the height of the handle and the knee-brace need adjusting.

The Turner should only be used on persons who can adequately weight bear and thus stand with support. The castors will run smoothly on any surface and they make the Turner easy to bring or push out of the way when not in use. It is folded by a knock-down function which allows for easy storage and transportation.



| | |
|--------------------|-----------------|
| Description | Item no. |
| Turner | 41 197 33 00 |



The carer takes hold of the upper handle with both hands and puts his or her foot on the pedal. The patient uses the lower handle to help him/her stand up while the carer uses his/her counterweight.




The Turner is turned round so that the patient ends up in the correct position in front of the new seat.



With one foot on the pedal, the carer uses his/her counterweight while the patient sits down.



Clean the product with a solvent-free detergent which has a pH of 5-9, or with a 70% disinfectant solution. Can be decontaminated at max. 85 °C if so required.

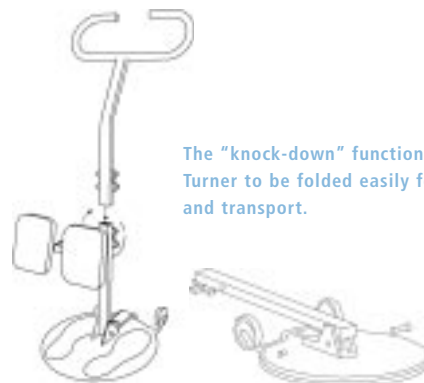

The "knock-down" function allows the Turner to be folded easily for storage and transport.